

Disposable Blood Dispenser

Disposable Blood Dispenser

The blood dispensers are non-sterile, single use medical devices specifically intended to provide a safe means for puncturing a closed collection tube and obtaining a drop of blood without need for removing the stopper, avoiding the hazardous release of airborne pathogens. The device produces a uniform drop of blood suitable for making a smear.



INSTRUCTIONS FOR USE

- ①  Open the packaging and get the device.
- ②  Hold tube in upright position, then insert device into stopper.
- ③  Invert and position device on target surface.
- ④  Gently press tube downwards until a drop come from the tip of the device.
- ⑤  Relax pressure after the instant drop touches target. For uniform drop size, slowly relax pressure to allow device to suck back excess specimen before removing device from target surface.
- ⑥  Dispose in a sharps medical waste container.

Disposable Bood Dispenser

Art.nr.	Box/ Case
366006	100/1500



Reference
1 Disposable Blood Dispenser IFU
Becton Dickinson Sweden AB, Marieviksgatan 25, 117 43 Stockholm, Sweden

[bd.com](https://www.bd.com)