
# STYLEVIEW® QUICK REFERENCE

**RANGE OF MOTION** 

20"

(508 mm)

# ergotron<sup>®</sup>

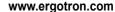

USA: 1-800-888-8458 Europe: +31 (0)33-45 45 600 China: 400-120-3051

Japan: japansupport@ergotron.com
Release Brake to move riser.



Loosen knob,



#### **ERGONOMICS**

## Working customize - to your size

(127 mm)

1 Set top of monitor screen about one inch below eye level - Release brake and lift or lower riser as needed.


20°

- 2 Tilt screen for comfortable viewing and to reduce eye and neck strain.
- 3 Pull keyboard tray forward and position mouse tray and mouse pouch on right or left, as needed.
- 4 Work with elbows bent at about 90° to minimize muscle strain.

5 If the riser moves up and down with difficulty, or if it drifts out of set position, consult the product

manual for adjustment information.

6 Stay in charge!
Powered carts should be
plugged into outlet as often as
possible to keep battery charged
and computer running.

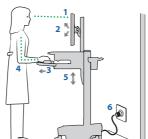

### Moving stow - before you go

1 During normal movement, release brake and lower worksurface to lowest position for optimal stability and unobstructed view.

±12

- 2 Tuck away open trays and return mouse, scanner and other accessories to their places.
- 3 Unlock both front casters.
- 4 Push cart from rear with elbows bent at about 90° to maximize control and minimize muscle strain.
- 5 Don't run out!

Before moving, make sure cord is unplugged from outlet and hooked to basket for safe travel. Remember, charge battery fully 100% every day!

